

Managing activity and exercise

What is the Activity feature?


While in Automated Mode, there may be times when the person living with diabetes would like less insulin automatically delivered. When the Activity feature is started, the SmartAdjust™ technology reduces insulin delivery and automatically sets the target glucose to 8.3 mmol/L for the time you choose.

When can the Activity feature be used?

During activities like sports, swimming, yard work, a walk in the park, or any other time when the glucose level tends to decrease.

How do I start the Activity feature?

1. Tap the menu button
2. Tap **ACTIVITY**
3. Enter the desired duration, then tap **CONFIRM**
4. Tap **START**

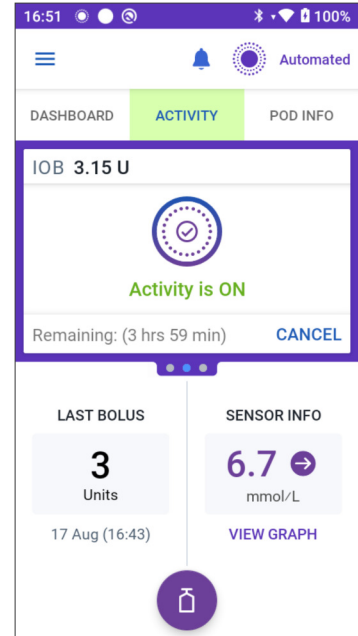

TIP!

This is when we like to use the Activity feature:
